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**IMPORTANT PRODUCT INFORMATION FOR:
DxH 500, PN B40601, Software Versions 1.0.0, 1.0.1, and 1.0.2**

July 2016

Dear Beckman Coulter Customer:

Beckman Coulter is sending you this letter regarding the possibility of specimen misidentification for the DxH 500 instruments. A specimen misidentification may occur, despite an active software prompt on the screen, when the following sequence of events occurs:

- Pending test orders exist on the Worklist.
- A new Specimen ID is added using the **Next** icon on the Sample Analysis – Patient Results screen or by scanning a specimen barcode label.
- The user leaves the screen before aspirating the new specimen.
- Upon returning to the Sample Analysis – Patient Results screen, the following prompt displays “*Worklist contains entries. Confirm next Specimen ID is correct before processing specimen*”.
- The Next Specimen ID field of the DxH 500 defaults to the first Specimen ID on the Worklist which may not be the newest specimen identified.
- The issue occurs when the user fails to verify the Specimen ID.

The issue does not occur when the Worklist has no pending test orders.

Always verify the Specimen ID displayed in the Next Specimen ID field matches the specimen to be processed before aspiration, as directed by the software prompt and the Instructions for Use PN B23922AA. Beckman Coulter is investigating this issue in order to provide a resolution.

Share this information with your laboratory staff and retain this notification as part of your laboratory Quality System documentation. If you have questions or concerns regarding this notification, call Beckman Coulter Customer Service at 1-800-526-7694 in Canada. Outside of Canada, contact your local Beckman Coulter Representative.

We apologize for any inconvenience to your laboratory.

Sincerely,

A handwritten signature in black ink that reads 'Caryl J. Campbell'.

Caryl J. Campbell B.S., MBA
Director of Global Technical Support Operations

PN B96702AA

Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc., in the United States and other countries.

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Cordialement,

A handwritten signature in black ink, appearing to read 'Caryl J. Campbell'.

Caryl J. Campbell B.S., MBA

Director of Global Technical Support Operations

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