## MVF / MERIVAARA France

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
**Dear user of Merivaara's Baby Crib Trolley 381**

We have noticed, that in some of the Merivaara's Baby Crib Trolleys' tightening screw of the adjusting bar (trendelenburg and anti-trendelenburg adjustment) have experienced opening of the threads. The reason for opening is that screw is 2 mm too short. The length of the screw is 20 mm. Then it was possible that the screw was not tighten long enough into nylock-nut. If this defect is not fixed and product is in continuous use there might be a minor risk of endangering patient's safety.

Baby Crib Trolleys that might have this defect were delivered between 1 May 2003 – 19 April 2006. Products delivered before 1 May 2003 or after 19 April 2006 don't have above described May defect. After 19 April 2006 delivered products do not have the above mentioned defect, because this defect was corrected in 19 April 2006, when we noticed it. Please see the pictures 1 and 2.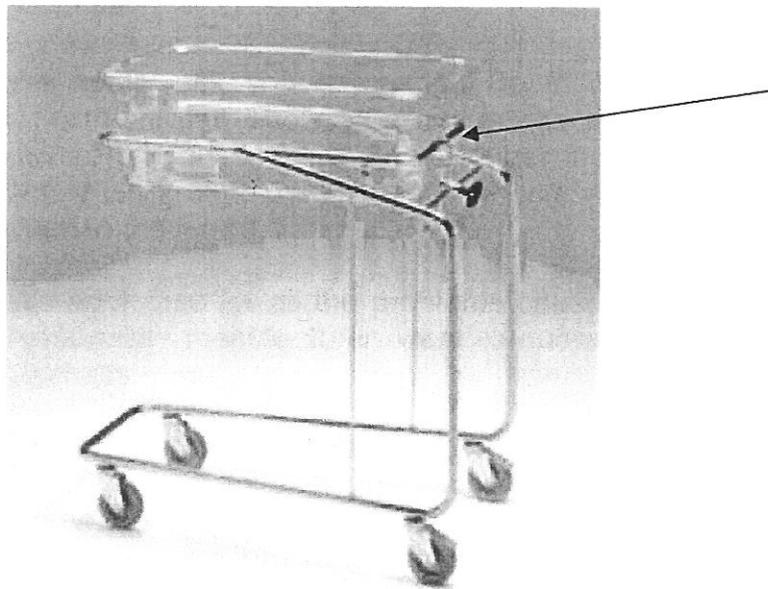

**Picture 1** Baby Crib trolley. The arrow shows the adjusting bar and its tightening screw.

**Merivaara Corp.**  
Puustellintie 2  
FI-15150 LAHTI, FINLAND  
Tel. +358 3 3394 611  
Fax +358 3 3394 6144


merivaara@merivaara.fi  
www.merivaara.com  
ID No. FI15046941  
Domicile Lahti

Nordea Bank Finland Plc  
Corporate Br., Lahti  
SWIFT code: NDEAFIHH


EUR 151930-128930  
USD 151952-11501  
SEK 151952-11519  
NOK 151952-11527

IBAN FI0315193000128930  
IBAN FI4215195200011501  
IBAN FI4115195200011519  
IBAN FI1915195200011527

30 May 2006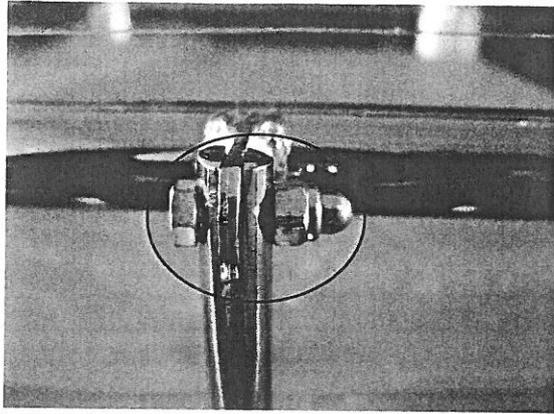

**Picture 2** shows the previous construction of adjustment bar (before 1 May 2003 delivered).

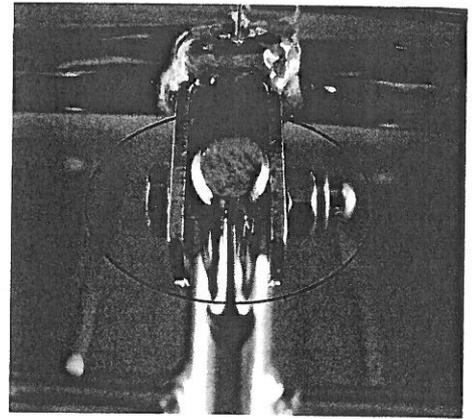**Picture 3** shows the construction, which might have the above mentioned defect, if they are delivered in between 1 May 2003 – 19 April 2006.

We kindly ask you to change Baby Crib Trolley's tightening screw of the adjustment bar. The new screw is attached to this letter. Please see enclosed picture 3. New screw is 2 mm longer than the old one. The length of the new screw is 22 mm. By changing the screw we make sure that the threads won't open in the future.

We apologise for all the problems caused to you by this procedure. We, as a responsible manufacturer, want to guarantee trouble-free and safe use of our products.

If you need further information, please contact our After Sales department.

Merivaara Corp./ After Sales  
Artem Kozmin  
tel. +358 3 3394 6181  
e-mail: artem.kozmin@merivaara.fi

Yours sincerely,

Merivaara Corp.

Jyri Johansson,  
Factory Manager

Merivaara Corp.  
Puustellintie 2  
FI-15150 LAHTI, FINLAND  
Tel. +358 3 3394 611  
Fax +358 3 3394 6144

merivaara@merivaara.fi  
www.merivaara.com  
ID No. FI15046941  
Domicile Lahti

Nordea Bank Finland Plc  
Corporate Br., Lahti  
SWIFT code: NDEAFIHH

EUR 151930-128930  
USD 151952-11501  
SEK 151952-11519  
NOK 151952-11527

IBAN FI0315193000128930  
IBAN FI4215195200011501  
IBAN FI4115195200011519  
IBAN FI1915195200011527