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---

### Siemens Healthcare Diagnostics Inc.

725 Potter Street  
Berkeley, CA 94710

877 229 3711  
[www.siemens.com/diagnostics](http://www.siemens.com/diagnostics)

Page 1 of 4

© 2008 Siemens Healthcare Diagnostics Inc. All rights reserved.

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\*\* Protease Inhibitors administered with low-dose ritonavir for pharmacological boosting.

Mutation Details for Protease Inhibitors	Evidence Basis
<i>Polymorphisms in protease in non-B subtypes may trigger rules of unknown clinical implications.</i>	
amprenavir (APV)/fosamprenavir (FPV) M46I indicates Resistance to amprenavir (APV)/fosamprenavir (FPV) (FPV-002)	IIa
APV/r or FPV/r ** I13V, M36I, M46I, I62V, L63P, and A71T indicate Resistance to APV/r or FPV/r ** (FPV-113)	IIa
M46I indicates Possible Resistance to APV/r or FPV/r ** (FPV-102)	IVa

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Resistance associated PR Mutations: I13V*, M36I*, I62V*, L63P*, A71T*	
Protease Inhibitors	Resistance Interpretation
amprenavir (APV)/fosamprenavir (FPV) APV/r or FPV/r **	No Evidence of Resistance <b>Resistance</b>
atazanavir (ATV) ATV/r **	No Evidence of Resistance
darunavir + ritonavir (DRV/r)	No Evidence of Resistance
indinavir (IDV) IDV/r **	No Evidence of Resistance
lopinavir + ritonavir (LPV/r)	No Evidence of Resistance
nelfinavir (NFV)	No Evidence of Resistance
saquinavir + ritonavir (SQV/r)	No Evidence of Resistance
tipranavir + ritonavir (TPV/r)	No Evidence of Resistance

\*\* Protease Inhibitors administered with low-dose ritonavir for pharmacological boosting.

Mutation Details for Protease Inhibitors	Evidence Basis
<i>Polymorphisms in protease in non-B subtypes may trigger rules of unknown clinical implications.</i>	
APV/r or FPV/r ** I13V, M36I, I62V, L63P, and A71T indicate Resistance to APV/r or FPV/r ** (FPV-113)	IIa

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