Dräger Medical AG & Co. KG, D-23542 Lübeck

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Sincères salutations  
Dräger Medical AG & Co. KG  
President Strategic Business Field Respiratory Care

Sebastian Kässner

# Supplement Paper to the Instructions for Use Oxylog 3000

This Supplement Paper must be kept with the Instructions for Use of the medical device.

This Supplement Paper updates the information in the Instructions for Use of the medical device in the following chapters:

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## en (9037171)

### For Your Safety and that of Your Patients

#### WARNING

To ensure proper ventilation, always consider the dead space of the total ventilation circuit when setting ventilation parameters, especially when applying small tidal volumes. When using tidal volume settings below 100 mL Dräger recommends constant monitoring of etCO<sub>2</sub> and SpO<sub>2</sub>.

#### Technical Data

Dead space including flow sensor	approx. 35 mL (reusable hose system)
	approx. 33 mL (disposable hose system)

## enUS (5704225)

### For Your Safety and that of Your Patients

#### WARNING

To ensure proper ventilation, always consider the dead space of the total ventilation circuit when setting ventilation parameters, especially when applying small tidal volumes. When using tidal volume settings below 100 mL Dräger recommends constant monitoring of etCO<sub>2</sub> and SpO<sub>2</sub>.

#### Technical Data

Dead space including flow sensor	approx. 33 mL (disposable hose system)
----------------------------------	--

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© Dräger Medical AG & Co. KG
**Draeger Medical, Inc.**
U.S.A.
Effective 2010-09: Dräger Medical AG & Co. KG changes to **Dräger Medical GmbH**