---

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Friadent GmbH

---

# Safety Information on ANKYLOS® Implant Driver C/X with Screwdriver

---

Dear user,

We regret having to inform you that an incident has been reported, where, owing to a chain of unfortunate circumstances, an instrument had been swallowed by a patient. The instrument could be successfully removed endoscopically.

The investigation of the case has shown that the instrument came off the placement head for the implant since the retention forces of the instrument had decreased. The incident occurred as the precautions prescribed in the instructions for use were not strictly observed.

For this reason we would like to direct your attention to the following matters addressed in the instructions for use for the Implant Driver in order to minimize any risk to the patient to the greatest extent possible:

1. **Only use the implant driver in combination with the ratchet.**
2. **If the circumstances of the treatment require the use of the implant driver without the ratchet, the instrument must be secured with a wire ligature or a tamponade must be applied in the pharyngeal zone to prevent swallowing.**

Additionally we would like to also remind you of the following instructions:

**Before using the instrument, check for correct operation and retention force of the instrument and make sure that the implant is securely retained.**

This precaution must be applied to all lots of the following instruments:

<b>3103 7551</b>	<b>Implant driver C/X with screwdriver, short</b>
<b>3103 7552</b>	<b>Implant driver C/X with screwdriver, medium</b>
<b>3103 7553</b>	<b>Implant driver C/X with screwdriver, long</b>

Please support us in applying these safety precautions and pass this information on to all the dental professionals in your area who use these instruments.

Friadent is using this incident for a design change and improvement of the product in order to permanently assure sufficient retention forces. The product will probably be available in mid 2011.

We appreciate your understanding. For any further questions, please contact our **implantology hotline at +49-621/4302 020** or your area sales manager.

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