**Bio-Rad
Laboratories**

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IMPORTANT PRODUCT INFORMATION: MonoFluo™ Kit Influenza - REFERENCE 52209, LOT 0M225U

This information is intended for the end user of this product

If you are not the end user, please forward this information to the appropriate laboratory personnel

Dear valued customer,

Following field complaints describing false negative results on Influenza B positive samples, the quality of the following product has been investigated:

- MonoFluo™ Kit Influenza
- Cat#: 52209
- Lot number: 0M225U
- Expiration date : 2012-07-15

Our quality control laboratory has confirmed the field results. It appears that a labelling error has occurred during manufacturing resulting in an inversion between reagent R1b (Monoclonal Ab for Influenza B) and R2 (negative control) in some vials.

According to insert instructions, each run must include a reference slide using positive and negative cells. Furthermore, for each patient, a negative control well must be tested in parallel of the specific antibody reagents R1a and R1b.

Therefore, in the inversion situation described above:

- Independently of patient status, the positive cell slide is falsely negative, and the run is invalidated.
- In the presence of an Influenza B positive sample, the negative control well is falsely positive, and the run is also invalidated.

As a consequence, we ask you to stop using the lot 0M225U and to destroy all the kits from this lot that you may still have in stock. A new lot is available in stock for exchange.

In order to proceed to the exchange, we ask you to fill the attached document.

May you have questions or need support on this subject, please do not hesitate to contact your local support.

Sorry for the inconvenience it might cause you.

Sincerely,

Marnes-la-Coquette, March 6th, 2012

Claude Giroud
Director Quality, Regulatory and Clinical Affairs

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PRODUCT REPLACEMENT

Form to return to your local customer service

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ACTIONS :

I am aware of information concerning the lot 0M225U kits MonoFluo™ Influenza Kit (Reference 52209) and have destroyed the relevant kits

Number of destroyed kits : _____

Date:

Laboratory Stamp and Signature

For any other further information or support, please contact your local customer service.