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INFORMATION PRODUIT IMPORTANTE

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Marnes-la-Coquette, September 17

IMPORTANT PRODUCT INFORMATION

This information is intended for the end user of this product
If you are not the end user, please forward this information to the appropriate laboratory personnel

Subject: Dengue NS1 Ag Strip (Ref. 70700) lots 2D0095, 2D0195 & 2D0295.

Dear Valued Customer,

Following field complaints describing **very faint test lines for supposedly negative patient samples**, the quality of the following product has been investigated:

- Ref: 70700 – Dengue NS1 Ag STRIP
- Lots :
 - 2D0095 (expiry date 28/02/2013)
 - 2D0195 (expiry date 28/02/2013)
 - 2D0295 (expiry date 30/03/2013)

Our quality control laboratory has confirmed the field results. For some negative samples, a very faint test line, described as "ghost", can be observed. According to customers who have experienced this type of results, these lines are clearly different from the image obtained with a true positive sample and have led to retest and confirmation request by another method.

If a false positive result was to be released based on such kind of line, the patient would be considered to be possibly infected by Dengue and would be monitored. Therefore this false positive result would lead to unjustified patient care but in no case would put patient at risk.

The lot file analysis has demonstrated that the following lots (starting with lot 2F0096) are not impacted by this quality issue. Testing has been done on 3 further lots and no false positive result has been observed.

As a consequence, we ask you to:

- Stop using kits from the 3 defective lots.
- Destroy the remaining kits.
- Fill the attached form to attest of product destruction and request for either replacement or credit note.

ANSM has been informed of this communication.

May you have questions or need support on this subject, please do not hesitate to contact your local support.

We apologize for the inconvenience, and remain at your disposal for any further information.

Please forward To Whom It May Concern.

Sincerely,

Claude Giroud

Director Quality, Regulatory and Clinical Affairs

PRODUCT REPLACEMENT
Form to return to your local customer service

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CUSTOMER INFORMATIONS

LABORATORY	
Undersigning Manager :	
Address :	
Telephone Number / Fax :	
Customer Account Number :	

ACTIONS :

- I have read information regarding the lots 2D0095, 2D0195 and 2D0295 of the device Dengue NS1 Ag Strip, Ref 70700

Number of kits to be exchanged: _____

Date:

Laboratory Stamp and Signature

For any other further information or support, please contact your local customer service.