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Support technique Carestream Dental

E-mail : europedental@carestream.com

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150 Verona Street, Rochester 14608, New York, USA

IRIX 70, IRIX 708, NOVELIX 6510 or ORAMATIC 558:

User guidance to identify if their product was made between November 1990 and November 1994 and if so, has the mandatory modification been made.

How to find year of manufacture

| | | | | |
|---|---|---|--|---|
|  |  |  |  |  |
| 1985 | 1987 | 1989 | Nov 1990 > Sept 1997 | Sept 1997 |

From November 1990 to December 1991, here is the serial n° visible on the generator:

| Type of generator | Production Start | Production End |
|-------------------|--------------------------------|------------------------------------|
| Irix 558 | Start Nov 90 (E03137) | Until end nov 92 (E03447) |
| Irix 708 | Start Nov 90 (G8665) | Until end nov 92 (G20315) |
| Oramatic 708 | Start Nov 90 (105560) | Until end nov 92 (113065) |
| Oramatic 558 | Start Nov 90 (J7578) | Until end nov 92 (J8734) |
| Irix 70 | | Until end August 97 |

In January 1992 an identification was done (2 letters) on the gray casting either on the right or the left side (see Fig 2). These letters were applied with a felt pen or a rubber stamp or by a punch which has widened the foundry. To read those letters remove the white plastic cover by removing one piece of gray metal and slide it backwards


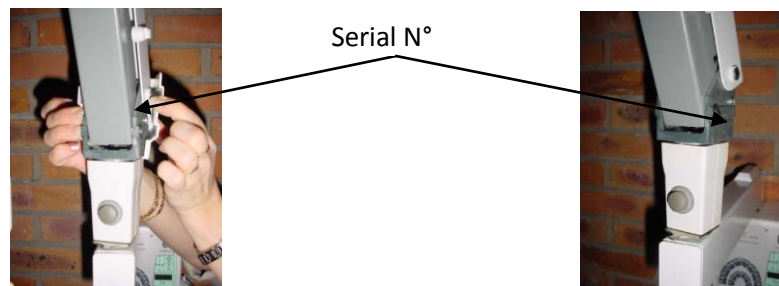
Fig 2


The signification of the 2 letters are for ex :

GK = November 1992
ID = April 1994
LI = September 1997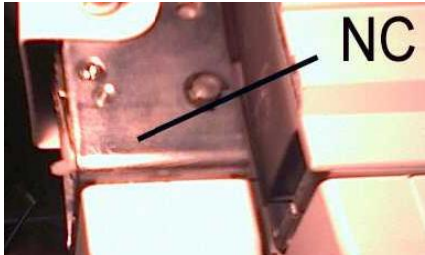

| | | | |
|---|------|---|-----------|
| G | 1992 | A | January |
| H | 1993 | B | February |
| I | 1994 | C | March |
| J | 1995 | D | April |
| K | 1996 | E | May |
| L | 1997 | F | June |
| | | G | July |
| | | H | August |
| | | I | September |
| | | J | October |
| | | K | November |
| | | L | December |

How to check if modification is done

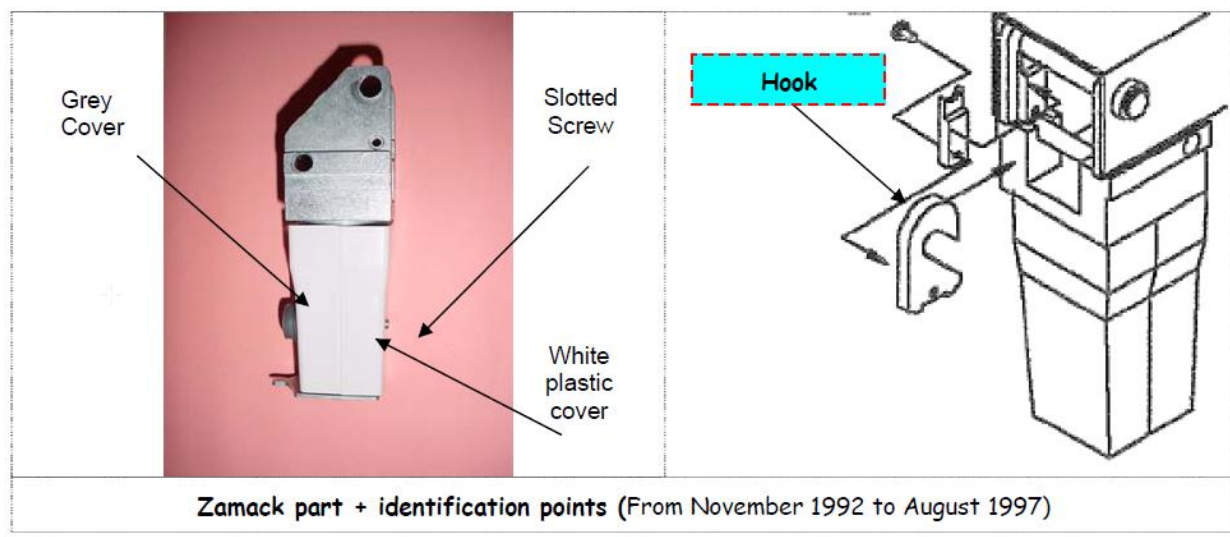If the hook is missing, modification has not been done contact Carestream Dental, if there are any difficulties, contact your dealer to complete the inspection or call Carestream Dental.

Contact Details for any information relating to this notice:

Carestream Dental Technical support team;

Email : europedental@carestream.com

Tel : 00 800 3456 6543

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Carestream Health Inc.