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Dear Customer,

Following the report by one customer of some barometric dispensing test failures on the system Elite™ observed with the positive control R4 and R5 of the Genscreen™ ULTRA HIV Ag-Ab (code 72388) lot 3F0674, our investigations results have confirmed a higher viscosity only for this lot. This lot event may lead to plate invalidation on systems ELITE™ and EVOLIS™.

A root cause analysis is underway to identify corrective action to be put in place to prevent the recurrence of this event in other lots.

The information provided by customer and our own investigations have shown no performance degradation of the reagent affected.

As a precautionary measure, we propose to customers **who have received this lot and who use the system Elite™, EVOLIS™, EVOLIS™ Twin Plus** to replace the R4 and R5 reagent bottles still available.

ANSM was informed of this communication.

You will find enclosed a form to fill out so we can proceed with sending reagent bottles R4 and R5 from another batch to replace those that you will destroy.

For any other further information or support, please contact your local customer service.

Thank you for the attention that you will bring to this mail and please accept, Madam, Sir, the assurances of our highest consideration.

Claude Giroud
Director Quality, Regulatory and Clinical Affairs

PRODUCT REPLACEMENT

Form to return to your local customer service

INVOLVED PRODUCT/LOT

Product Name : Genscreen™ ULTRA HIV Ag-Ab - R4 and R5	
Réf : Reagent R4 and R5 in the lot 3F0674 of Genscreen™ ULTRA HIV Ag-Ab kit (code 72388),	Lot : 3F0256

CUSTOMER INFORMATIONS

LABORATORY	
Undersigning Manager :	
Adress :	
Téléphone Number / Fax :	
Customer Account Number :	

ACTIONS :

- I have read information regarding the lot 3F0256 bottles positive control (R4 and R5 reagents) present in kits 3F0256 lots of Genscreen™ ULTRA HIV Ag-Ab (code 72388).

Number of vials to be exchanged: _____

Date:

Laboratory Stamp and Signature

For any other further information or support, please contact your local customer service.