Vienne, Juin 2016

ULTRASTOP pro med.

Madame, Monsieur,

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ANTIFOG-AGENT

INGREDIENTS / PROPERTIES

ULTRASTOP „pro med.“ contains a special combination of surface-active substances in an alcohol/water matrix (ethanol, purified water, < 5 % surface-active substances).

ULTRASTOP „pro med.“ is a high-quality sterile anti-fogging agent for medical lenses.

Examination mirrors, endoscopes and similar devices for aseptic surgical use such as microscope oculars and lenses can also be easily, simply and reliably protected from fogging.

The special property of **ULTRASTOP „pro med.“** is that it reliably prevents fogging on the surfaces of medical lenses. Surface-active substances in a special combination form an absolutely clear and transparent film on the treated surfaces. This extremely thin film prevents the formation of droplets when water vapour becomes condensed; thus it prevents fogging of the surface.

AREAS OF USE

ULTRASTOP „pro med.“ is especially suitable for:

- ♦ use in **strictly aseptic endoscopy**, which has to be performed with sterile apparatuses in visceral cavities (e.g. laparoscopy, thoracoscopy, arthroscopy etc.) as well as for
- ♦ all medical examination mirrors and lenses (e.g. larynx and epipharynx mirrors, stroboscopes, dental mirrors, colonoscopes, etc.).

Medical examination mirrors, endoscopes and similar devices used in aseptic surgery as well as microscope oculars and lenses can be easily, simply and reliably protected from fogging. Pre-warming the optics is no longer necessary which is time- and labour-saving for the doctor.

Note: **ULTRASTOP „pro med.“** can be used on glass and nearly all synthetic materials. If other coatings have been used or if the coating is of doubtful resistance, please contact the manufacturer or the retailer of the object to be treated.

INSTRUCTIONS FOR USE

Apply **ULTRASTOP „pro med.“** directly or by means of an **ULTRASTOP „pro med.“**-moistened sterile gauze to the disinfected or sterilised dry and clean surface and spread evenly. Leave it to act for a short time and after evaporation, if necessary, gently polish with sterile non-threading gauze.

ULTRASTOP „pro med.“ contains no glycerine or similar substances and, if used properly, does not smear. However, if too much is used the surface may appear cloudy. In this case the surface should be cleaned with distilled water or sterile gauze, and the above mentioned procedure should be repeated.

Accurate aseptic use: Desinfect aluminum cap and rubber pad (latex-free) before each use. To remove the solution use only sterile syringes. Do not leave the needle in the rubber stopper!

⚠ Not for injection! Highly flammable!
Keep out of the reach of children!

STORAGE CONDITIONS: Do not store above 25 °C. Protect from light.

SHELF-LIFE: **ULTRASTOP „pro med.“** is considered to be auto-sterile. It can be used, under the stated conditions, until the expiry date printed on the package.

AVAILABLE IN THE FOLLOWING FORMS

ULTRASTOP „pro med.“, 30 ml vial (**sterile**)

ULTRASTOP „pro med.“, 5 ml Monodose (**sterile**):

1 x 5 ml + Application kit, 20 x 5 ml + Application kits, 30 x 5 ml Clinic

ULTRASTOP spray, 15 ml

ULTRASTOP bottle with dropper pipette, 25 ml

Manufacturer: MoNo chem-pharm Produkte GmbH, LeystraÙe 129, A-1200 Vienna - Austria

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Produttore: MoNo chem-pharm Produkte GmbH, Leystraße 129, A-1200 Vienna
Distribuzione: SIGMAPHARM Arzneimittel GmbH, A-1200 Vienna