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Attention:

Customers who received one or more of the products listed below.

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Affected are all feet and knee joints with article number 1E2 ProCarve Foot Standard, 1E2=1 ProCarve Foot Firm and 3R2 ProCarve Knee Joint, that have been distributed since sales of these products started (all serial numbers).

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You can identify the articles on the basis of the catalog number. This number can be found on the back for the feet and for the knee joint (see picture above).

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The ProCarve components are recommended for use for sports like skiing, snowboarding, water-skiing, wakeboarding.

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We were advised that the prosthetic components may restrict the user in their movements due to the weight, when using the prosthetic components in water. This may result in the user being pulled down and the risk of drowning exists.

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Until now no feedback from the market came to our attention that a hazardous situation occurred due to the described circumstances.

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Otto Bock HealthCare GmbH has decided to inform customers and users about this issue voluntarily and precautionary.

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Check if there are articles with article number 1E2, 1E2=1 and/or 3R2 on your stock. If yes, deposit the attached flyer in addition to the product.

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Contact immediately all users. Inform the user about the importance **to use a flotation device when playing water sports** (see attached flyer).

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Complete the attached Customer Reply form and send it back to the mentioned contact person **until December 31st, 2016**.

Transmission de cette information de sécurité urgente / Transmission of this Urgent Field Safety Notice:

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This notice needs to be passed on to all those who need to be aware of this situation within your organisation.

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Please pass on this notice to all persons / organisations to which the products were given and are affected by this action.

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We apologize for any inconvenience this action may cause.

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Otto Bock HealthCare GmbH a décidé d'informer volontairement de ce problème les clients et les utilisateurs.