Marnes-la-Coquette, le 15 novembre 2011

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Dear Customer,

We would like to inform you that Bio-Rad has received the attached product recall notice (Appendix A of this letter) from Hamilton Corporation. This notice concerns several batches of Microlab FAME Disposable Syringes used in the Elite and FAME Systems.

Hamilton has initiated this action because of a potential defect leading to increased occurrence of error message DIS541 ("Invalid distribution"). The lots listed in Hamilton's notification include: 31111, 50711, 50911 and 51111. The investigation conducted by Bio-Rad has found that upon occurrence of the DIS541 error, the results for the microplate being processed are rejected by the software. Hence, no results are produced.

Upon receipt of this communication, we ask that you:

1. Refrain from using the affected lots.
2. Attentively read the product recall notice from Hamilton.
3. Return the completed Return Receipt attached as Annex B to this letter and complete the Bio-Rad acknowledgment form.

Bio-Rad will contact you to plan the exchange of the lots 31111, 50711, 50911 and 51111 syringes still in your possession.

We inform you that AFSSAPS was informed of this.

For any other further information or support, please contact your local customer service.

Thank you for the attention that you will bring to this communication and please accept, Madam, Sir, the assurances of our highest consideration.

Claude Giroud

Director Quality, Regulatory and Clinical Affairs

Annex A – Hamilton Notification (Text)

HAMILTON Bonaduz AG
P.O. Box 26
CH-7402 Bonaduz, Switzerland
Phone: +41 81 660 60 60
Fax: +41 81 660 60 70
E-mail: itechsupport@hamilton.ch

Microlab® F.A.M.E. - Disposable Syringe - Part Number 147692

It has been reported to HAMILTON that some lots of Microlab F.A.M.E. Disposable Syringes have caused an increase in errors DIS541 ("improper dispensation") during routine user operation.

Investigations have revealed that the barrel-plastic used at begin of this year was out of its specifications. This might lead to an increase of the error DIS541 ("improper dispensation").

We at HAMILTON have taken this situation very seriously and we have re-tested all potentially affected syringe lots with our archived samples of each lot. These tests have shown that the following lots are affected by this material failure:

30311; 50411; 30511; 50611; 50711; 50811; 30911; 50911; 51011; 31111; 51111; 31211; 51211; 31311; 51311

According to the current state of knowledge the affected lots do not constitute a hazard to human health. However, we at HAMILTON understand that an increase of an error DIS541 during routine use is not acceptable for our customers.

Therefore, we will replace all Microlab F.A.M.E. Disposable Syringe Sets with the lot numbers above.

Corrective Action :

The manufacturer of the syringes immediately evaluated complained syringes from the field and has verified if all production processes are still working correct. Investigations of the material finally have shown the reason mentioned above.

HAMILTON and the syringe manufacturer have implemented additional quality control checks for the material of the syringe to ensure continued quality in manufacturing.

HAMILTON Bonaduz AG apologizes for any inconvenience this situation may have caused. We would like to assure you that all necessary steps have been taken to maintain a continued supply of high quality disposable syringes.

For the replacement of the affected disposable syringe lots, please contact our Customer Service Department.

Please do not hesitate to contact me if you have any further questions.

With kind regards,

Tscharner Georg
Product Manager OEM
HAMILTON Bonaduz AG
e-mail: gtscharner@hamilton.ch

Annex B - PRODUCT REPLACEMENT

Form to return to your local customer service

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ACTIONS :

- I have read information regarding the Concerto Syringes 20 ml (product code 91371) Lots 31111, 50711, 50911 et 51111.

Number of syringes to be exchanged: _____

Date:

Laboratory Stamp and Signature

For any other further information or support, please contact your local customer service.