INFORMATION PRODUIT IMPORTANTE

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Marnes-la-Coquette, September 13, 2012

IMPORTANT PRODUCT INFORMATION

This information is intended for the end user of this product
If you are not the end user, please forward this information to the appropriate laboratory personnel

Subject: Dengue NS1 Ag Strip (Ref. 70700) lots 2D0095, 2D0195 & 2D0295.

Dear Valued Customer,

Dear all,

Following field complaints describing incoherent susceptibility profile for some clinical strains revealed by a colour change to yellow of cefixime well leading to a resistant clinical categorisation when all other cephalosporins of the microplate remained susceptible, investigations were done on the following product:

- Ref: 66781 – SIR ENTERO II
- Lot : **2G2123**
- Expiry date: 07/30/2013

Our quality control laboratory has confirmed the false resistance results obtained by some end-users. Stability studies have not demonstrated any raw material degradation overtime. A human mistake, leading to inversion of cefalotine and cefixime wells, has caused this failure. Correctives actions are being implemented to avoid such failure. In this case, the risk is to limit the choice of antibiotics available to treat the patient by excluding cefixime whereas it may be appropriate.

As a consequence, we ask you to:

- Stop using kits from the 3 defective lots.
- Destroy the remaining kits.
- Fill the attached form to attest of product destruction and request for either replacement or credit note.

If you are not the final user of above mentioned lots of SIR Entero II kits, please take necessary actions to forward this information to your own customers.

ANSM has been informed of this communication

May you have questions or need support on this subject, please do not hesitate to contact your local support.

We apologize for the inconvenience, and remain at your disposal for any further information.

Please forward To Whom It May Concern.

Sincerely,

Claude GIROUD

Director Quality, Regulatory and Clinical Affairs

PRODUCT REPLACEMENT
Form to return to your local customer service

INVOLVED PRODUCT/LOT

Product Name : SIR Entero II	
Réf : 66781	Lot : 2G2123

CUSTOMER INFORMATIONS

LABORATORY	
Undersigning Manager :	
Address :	
Telephone Number / Fax :	
Customer Account Number :	

ACTIONS :

I have read information regarding the lots **2G2123** of the device

Number of kits to be exchanged: _____

Date:

Laboratory Stamp and Signature

For any other further information or support, please contact your local customer service.