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URGENT: FIELD SAFETY NOTICE
Medical Device Safety Advisory Notice and Product Recall

Date: 11 April 2013

BD Product Reference	Description
300223	BD Plastipak™ 50ml Luer Lok Syringe – Non Sterile
300865	BD Plastipak™ 50ml Luer Lok Syringe - Sterile
300869	BD Plastipak™ 50ml Luer Lok Syringe Amber - Sterile

For the Attention of:

- **Distributors supplying BD Plastipak™ 50ml Luer Lok Syringes**

Please be advised that this Field Safety Notice supercedes any prior BD communication regarding this issue (please discard any prior communications that you may have received).

Product Advisory Notice

BD launched a new syringe stopper design in April 2012. Syringes with the new stopper design (reference enclosed picture) may exhibit higher plunger forces when used in pumps.

The higher pump forces may increase the likelihood of triggering a false occlusion alarm for the current setting of occlusion level for some pumps. Please contact your BioMedical Engineering Department (EBME) to assess the pump and syringe system and make adjustments if necessary.

Becton, Dickinson U.K. Limited

Registered in England: 00852702 Registered Office: The Danby Building, Edmund Halley Road, Oxford Science Park, Oxford, Oxfordshire OX4 4DQ

The Danby Building
Edmund Halley Road
Oxford Science Park
Oxford OX4 4DQ
Tel: 01865 748844
Fax: 01865 717313
www.bd.com

Product Recall

While most of the syringes in the product distribution/usage chain have the new design stopper, there may be limited inventory of syringes with the previous design stopper. If syringes with the previous design stopper are used in syringe pumps that have been adjusted for the new stopper, there may be a delay in initiating an occlusion alarm. In order to minimize the potential for this occurrence, **BD is recalling syringes with the previous design stopper.** The recalled product (with the previous stopper design, i.e. larger stopper) can be identified as detailed in appendix 1. Please note that only previous design (larger stopper) product is being recalled.

YOU NEED TO TAKE THE FOLLOWING ACTIONS:

1. Please check your inventory for the three BD products listed. Lot numbers are identified in appendix 1. Return any affected products to BD in accordance with the instructions on the attached packing list.

BD Product Reference	Description
300223	BD Plastipak™ 50ml Luer Lok Syringe – Non Sterile
300865	BD Plastipak™ 50ml Luer Lok Syringe - Sterile
300869	BD Plastipak™ 50ml Luer Lok Syringe Amber - Sterile

2. Please ensure that a copy of this letter is provided to any organizations to which the affected devices have been transferred or distributed.
3. Please complete the Customer Advisory Notice and Recall Response Card below and return by fax immediately to the number on the Customer Response Card.

Note: THIS FORM MUST BE COMPLETED AND FAXED TO BD MEDICAL.

4. If you have any questions regarding this communication, please contact the following telephone number:

General Enquiries: 01865 781517

EBME / Technical Queries: 01204 476090

Becton, Dickinson U.K. Limited

Registered in England: 00882702 Registered Office: The Danby Building, Edmund Halley Road, Oxford Science Park, Oxford, Oxfordshire OX4 4DQ

The Danby Building
Edmund Halley Road
Oxford Science Park
Oxford OX4 4DQ
Tel: 01865 748844
Fax: 01865 717313
www.bd.com

BD

Helping all people
live healthy lives

Please accept our apologies for the inconvenience caused by this advisory and recall notice. We know that you share in our desire to provide superior quality products and services to both our customers and their patients.

Yours sincerely,

A handwritten signature in black ink, appearing to read 'Julie Cotterell'.

Julie Cotterell
Marketing Manager
BD Medical – Medical Surgical Systems

BD, BD Logo and trademarks are property of Becton, Dickinson and Company © 2013

Becton, Dickinson U.K. Limited

Registered in England: 00852702 Registered Office: The Danby Building, Edmund Halley Road, Oxford Science Park, Oxford, Oxfordshire OX4 4DQ

The Danby Building
Edmund Halley Road
Oxford Science Park
Oxford OX4 4DQ
Tel: 01865 748844
Fax: 01865 717313
www.bd.com


BD

Helping all people
live healthy lives

BD Plastipak™ 50ml Luer Lok Syringe Pictures

PREVIOUS DESIGN (being recalled)

CURRENT DESIGN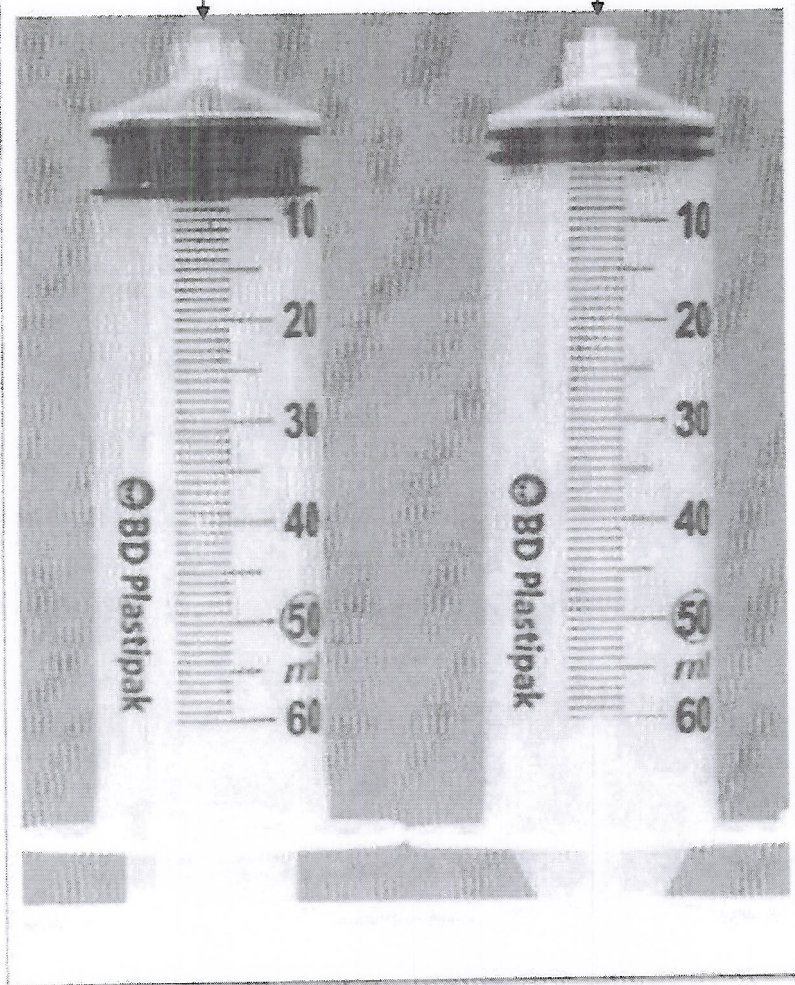

Becton, Dickinson U.K. Limited

Registered in England: 00852702 Registered Office: The Danby Building, Edmund Halley Road, Oxford Science Park, Oxford, Oxfordshire OX4 4DQ