---

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**Transmission of this Field Safety Notice:**

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Laure Landucci  
Quality and Regulatory Affairs Manager

<u>Advices regarding the safe use of the footrest /</u> <u>Conseils d'utilisation du repose pieds allégé</u>	
Date:	25/07/2013
Supersedes Bulletin N°:	-
Product:	PLATINUM
Author:	Jean-Philippe Dross
Compliance Category	<input type="checkbox"/> <b>Amelioration Process:</b> Identifies documentation changes or any administrative information that may be beneficial to some but not necessarily all customers. Accomplishment is at customer's discretion.  <input type="checkbox"/> <b>Maintenance:</b> Identifies maintenance guidelines or best practices for reliable equipment operation.  <input type="checkbox"/> <b>Corrective Action:</b> Identifies the need for action to correct a condition that, if left uncorrected, may result in reduced equipment reliability, efficiency or prevent of material damages. Compliance may be required within a specific operating time.  <input checked="" type="checkbox"/> <b>Safety:</b> Failure to comply with this Service Bulletin could result in personal injury. Compliance must be done in an urgent operating time.
Timing Code	<input type="checkbox"/> <b>Optional or Not applicable</b>  <input type="checkbox"/> <b>Mandatory / Next Schedule visit</b>  <input checked="" type="checkbox"/> <b>Mandatory / Urgent</b>

# Service Bulletin

SB13-0008

## ABSTRACT / OBJET:

A use error while setting up the footrest has been observed that may result in a patient injury.

This service bulletin reminds the advices to follow to check the correct footrest locking. This must be forwarded to every operator who could use this kind of footrest.

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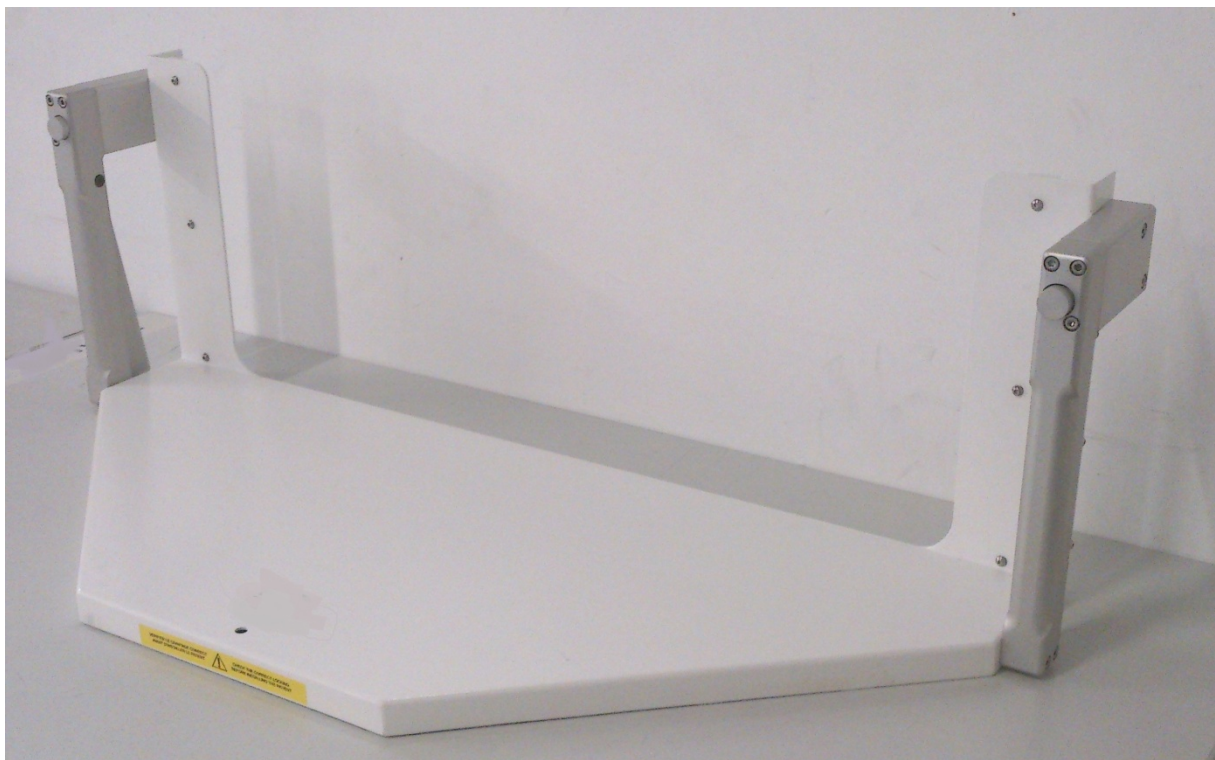
## APPLICABILITY / DOMAINE D'APPLICATION:

For all PLATINUM models using a light footrest model with reference Z6F35081 as on following picture:

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### DESCRIPTION:

To ensure optimal safety, please observe the following operating instructions:

- Establish the footrest to the desired height detent
- Release the handles and make sure the 2 latches (left and right handles) are properly locked. The lock is correct when the grooves of two pins are visible.

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The groove of the pin is visible: **locked** foot rest.

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The groove of the pin is not visible : **not locked** foot rest.

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