ALCIS SARL

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*In compliance with its quality system certified ISO 13485, Alcis has tested some Adipose Tissue Treatments Kits REF 2054-KTA2 after accelerated aging. For one product among three we observed a insufficient tensile resistance value of the external packaging. The internal packaging of the device present a sufficient and compliant tensile strength after aging.*

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*Consequently, there is a potential risk in case of use of these products, if the external packaging remains opened or damaged. So, in collaboration with ANSM, the Alcis Company has decided to inform his customer of the risk detected and on the importance to observe the recommendation of the IFU (see extract below).*

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*After internal analysis we have concluded that this observation is limited to product REF 2054-KTA2 of batch 14/326.*

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## WARNINGS AND RECOMMENDATIONS

It is essential to observe the recommendations set out in the instructions for use. The kit must only be used by a surgeon qualified in cosmetic and reconstructive skin procedures. Open the kit and lay out its contents in a sterile environment, particularly the cannula (C) that is used throughout the protocol. Do not use the kit after the expiry date indicated on the packaging or if all of the 18 components are not present. During the procedure, standard rules and procedures in the event of asepsia must be observed. The wearing of safety goggles is compulsory when using the kit. The kit must be stored in a dry environment at a maximum temperature of 25°C. It must be protected from light and handled with care to avoid damage from impact. Handle the components of the kit with care. Before beginning the procedure, inspect the kit and its packaging before beginning the procedure in order to verify that it has not been damaged in any way: do not use the kit if the packaging is open, damaged or flawed so as to affect the sterility of the kit. Do not use the kit if one of its components is damaged.

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